


How to collect urine


outer tube

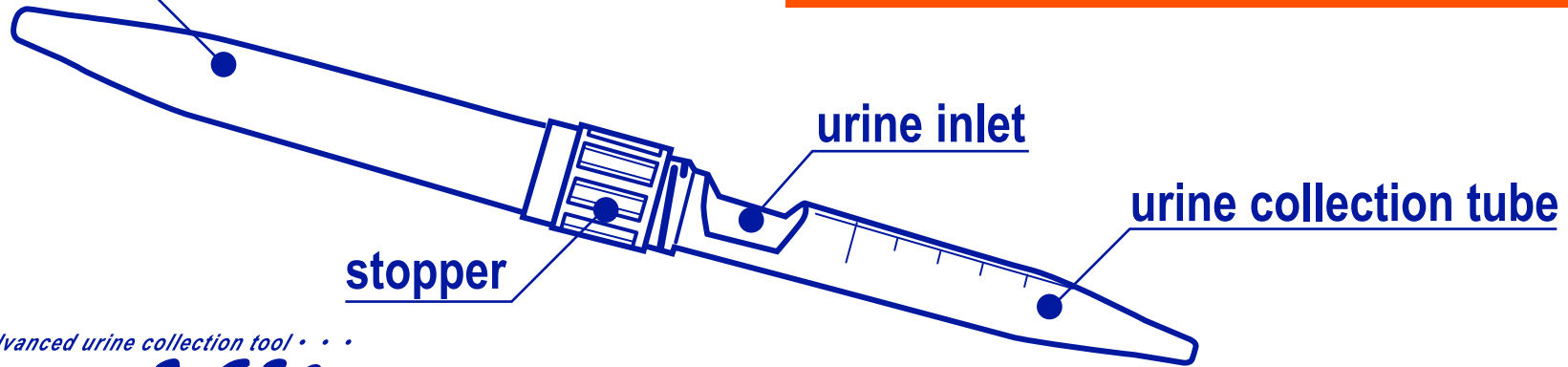

• • • innovative & advanced urine collection tool • • •


Harn Kit

① For easy urine collection, displace the position of outer tube.

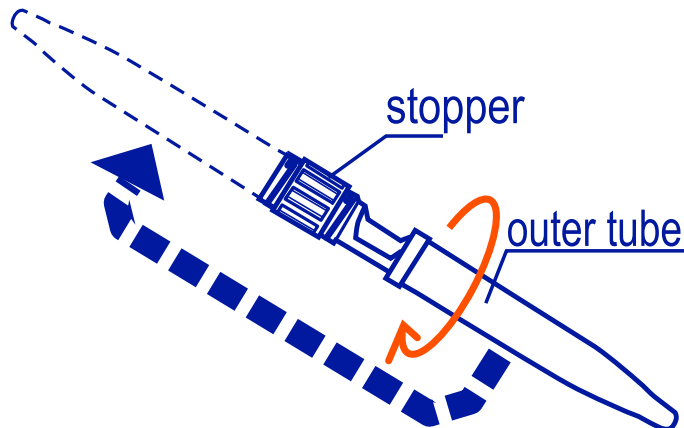


Turn the outer tube to the **left and withdraw**, then **fix** it at the **other end of the stopper**.

② Hold outer tube and collect urine.

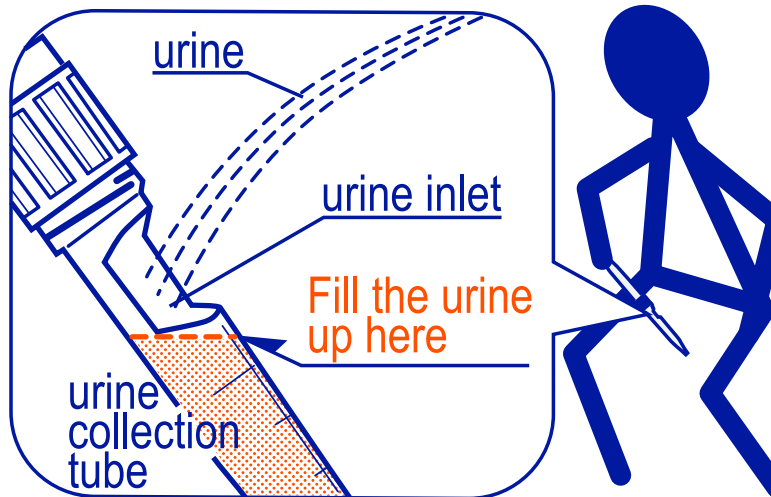

Pour urine into the urine inlet, while urination and **fill the tube with full of urine**.

③ Return the outer tube to the original position.

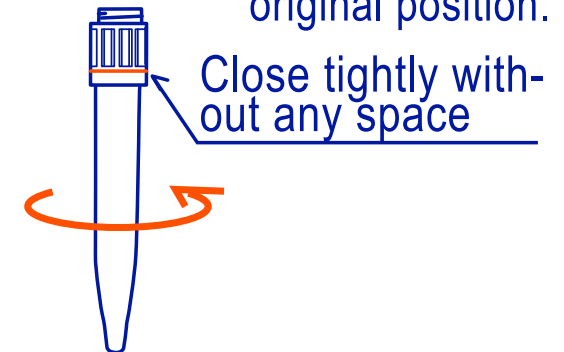

Return the outer tube to the original position to cover urine collection tube, then turn it to the right to tighten firmly. **Confirm no leakage of urine**.